
Fever FRIENDZ®

Stick-On Fever Indicators

Fever-Friendz® is a kid friendly temperature indicator that allows you to easily monitor your child's temperature for up to 48 hours.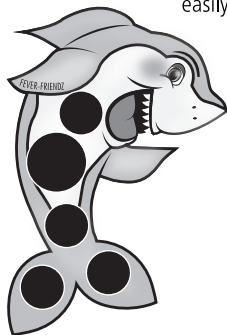

How does Fever-Friendz® work?

Fever-Friendz® uses encapsulated liquid crystal technology to indicate the user's body temperature in 1 degree Celsius increments.

Fever-Friendz® is the ideal solution for continuous temperature monitoring because it is:

- Fast, effective and safe
- Accurate within 0.5° C
- Visible at a glance
- Hypoallergenic

Using Fever-Friendz®


Step 1: Fast, Safe and Gentle Application

Simply peel Fever-Friendz® off the backing and apply directly to the skin. For optimal accuracy place on the forehead. It can also be placed under the armpit, on the chest, or between the shoulder blades.

Step 2: Visible Readings in Seconds

Within seconds Fever-Friendz® light up indicating N (for normal), 37°, 38°, 39°, or 40° (and over). The temperature reading corresponds to the highest figure that appears green. If green does not appear, the temperature will be mid-way between that indicated by the tan and the blue spots. Normal axillary temperature range is 36.1° C to 37.2° C.

Fever-Friendz® will adhere for up to 48 hours, depending on skin type, placement or activity. Removing and repositioning reduces adhesion.

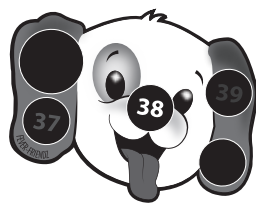Temperature indicated: 38° C

Step 3: Easy, Painless Removal

When continuous monitoring is no longer necessary, simply remove Fever-Friendz® and discard. **DO NOT REUSE.**

WARNING! Choking hazard. Store out of reach of small children. Recommended for children 12 months and older. Fever-Friendz® is a fever indicator. Always use a thermometer to confirm temperature reading. Use of the fever indicators is not intended as a substitute for a consultation with your healthcare professional.

For readings you can trust, discover the complete line of Physio Logic® thermometers.

Kids and parents will love Fever-Friendz®, the convenient, easy to use temperature indicator for continuous fever monitoring. For those times when you want fast, precise readings, use one of our Accuflex® or Quick Scan™ digital thermometers.

Feature	Fever-Friendz® Stick-On Fever Indicators	Accuflex® Flexible Digital Thermometers	Quick-Scan™ Ear and Forehead Thermometer
Speed	Visible at a glance	5 or 10 seconds*	0.1 seconds
Accuracy	± 0.5°C	± 0.1° C / 0.2° F	± 0.2° C / 0.4° F
Comfort	Gentle, safe and kid-friendly	Gentle-flex tip adds comfort & safety	Scan from ear or forehead
Hygiene	Disposable	Washable	Washable
Convenience	48 hours continuous monitoring	6 times faster than conventional digital thermometers	No probe covers required

* Average clinical test results for Accuflex 10: 10.2 seconds rectal, 15.2 seconds oral, 20.8 seconds under the arm. Clinical results range from 9.75 to 28.0 seconds. Average clinical test results for Accuflex 5: 5.6 seconds oral, 10.2 seconds rectal, 6.9 seconds under the arm.

Visit www.amgphysiologic.com or www.fever-bugz.ca for more product information.

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