Prepare the skin:

1. Each area to be treated should be washed with soap and water, rinsed and dried prior to application of the gel pad.

Applying the self-adhesive gel pads:

1. Gently remove the plastic storage liner by peeling it off from the pad.
2. Apply gel pad only to unbroken, non-irritated, healthy, intact skin.
3. Apply the gel pad on targeted treatment zone as recommended by your physician, therapist or medical professional by applying the center of the gel pad first and smoothing outward.
4. Gel pad must be placed firmly and evenly to skin and have good contact between the skin and the gel pad.
5. Before connecting the TheraHeat™ TENS unit to the gel pad, ensure that the unit is turned OFF.
6. Attach the TheraHeat™ TENS unit to the gel pad by aligning the gold and silver connectors
7. Adjust the treatment zones as recommended by your medical professional.

Removal and storage of gel pads:

1. Turn off the TENS unit before removing the gel pad.
2. Remove the TheraHeat™ self adhesive gel pad from skin by peeling from pad edge, and in order to maximize their lifetime, we recommend placing them on to their original plastic storage liner after each use, and keep them at room temperature in a zip lock bag.
3. If adhesive becomes over-saturated, place gel pad face up at room temperature and let dry until gel regains its tackiness. If the gel pad appears dry, add a few drops of water and let dry until gel regains its tackiness.

Notes:

- *The lifetime of your gel pad will vary depending on your skin condition, skin preparation before application, type of stimulation, storage conditions and climate.*
- *Self adhesive gel pads should be replaced when they lose their adhesiveness or when you notice a change in the intensity of the stimulation.*
- *If skin irritation occurs, discontinue use and consult your healthcare professional.*

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