**(Model / Modèle 740-744)**

# Deluxe

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**proactive™**  
Physiotherapy • Physiothérapie

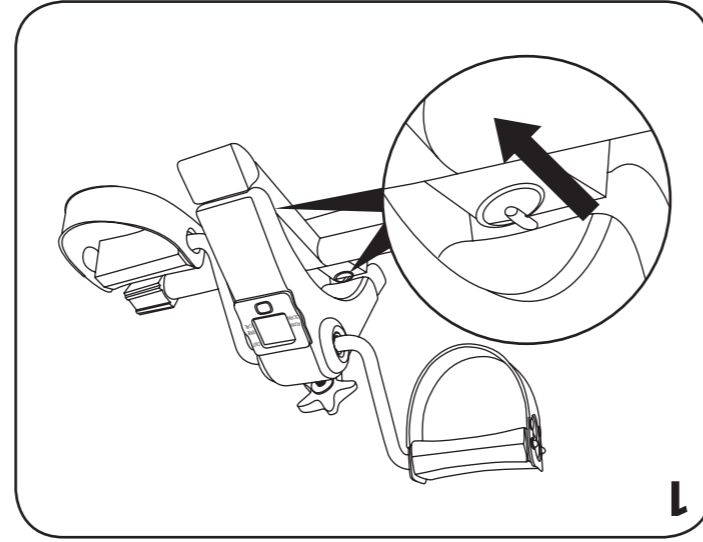
**This Proactive™ Deluxe Pedal Exerciser** is a compact exercise bike that stores conveniently out of the way when not in use. Use it regularly to stimulate blood circulation and increase muscle strength: it is a great way to enjoy a moderate cardiovascular workout. Place it on a tabletop to exercise your arms or on the floor to enjoy a bicycle-type exercise from your chair.

**WARNING!** Metal frame might get hot when used for a long period of time. Use the knob to lift or move the exerciser.

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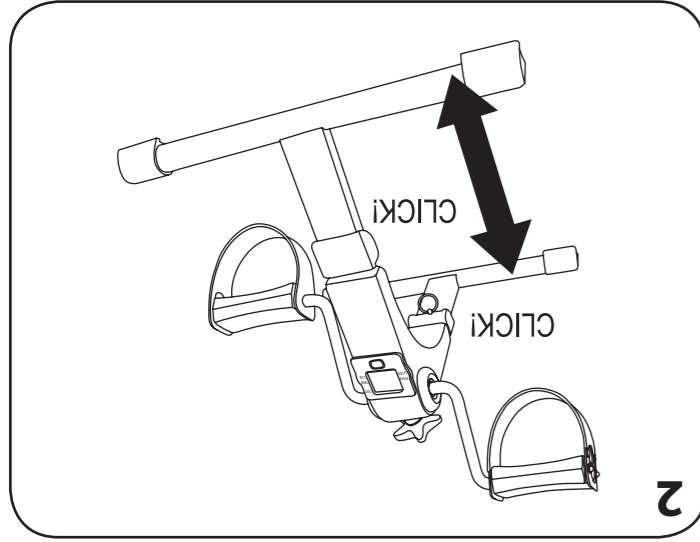
### 1 year limited warranty

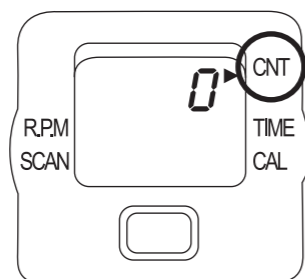
AMG Medical Inc. warrants this product to be free from defects in materials or workmanship for a period of one (1) year from the date of purchase. This warranty is valid for the original purchaser only. Any alterations, misuse or accidental damage voids this warranty.

Questions? Comments? / Questions? Commentaires?  
[www.amgmedical.com](http://www.amgmedical.com) • 1-800-363-2381

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To turn on the display, press once on the red button or start pedaling. The display automatically shuts off after 4 minutes of inactivity. All of the display functions are activated as soon as you begin pedaling and pause when you stop. To toggle between them, press the red button until the arrow points to the desired function.

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

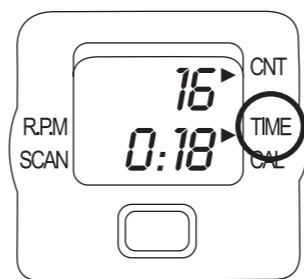
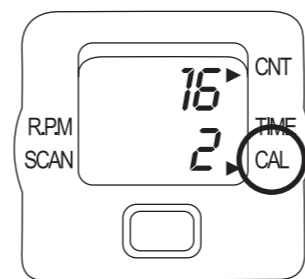
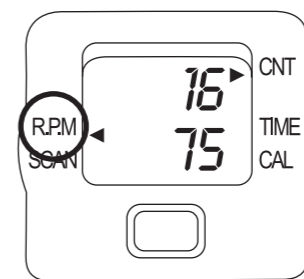
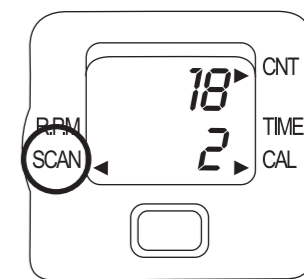
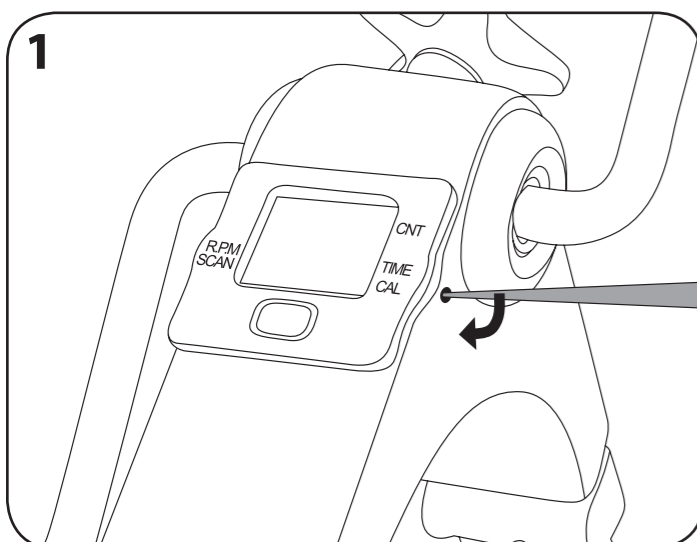
Insert a sharp object in the opening on the screen's right and push it up to unlock the screen's faceplate.

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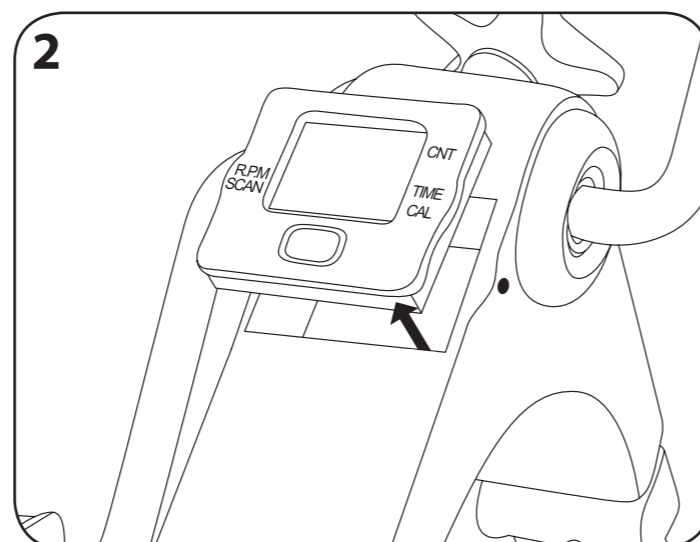
Remove the screen unit from its socket.

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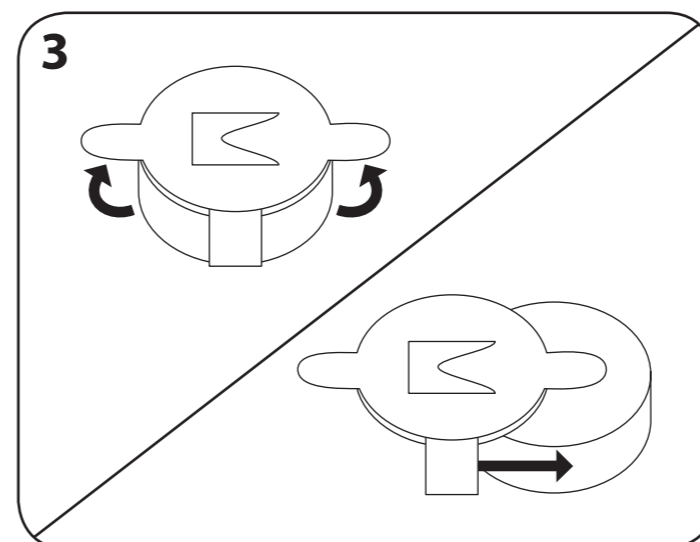On the back of the screen unit, lift up the battery tabs and slide out the battery. Insert a new 1.5V alkaline AG13 battery, positive side down, and close the tabs.

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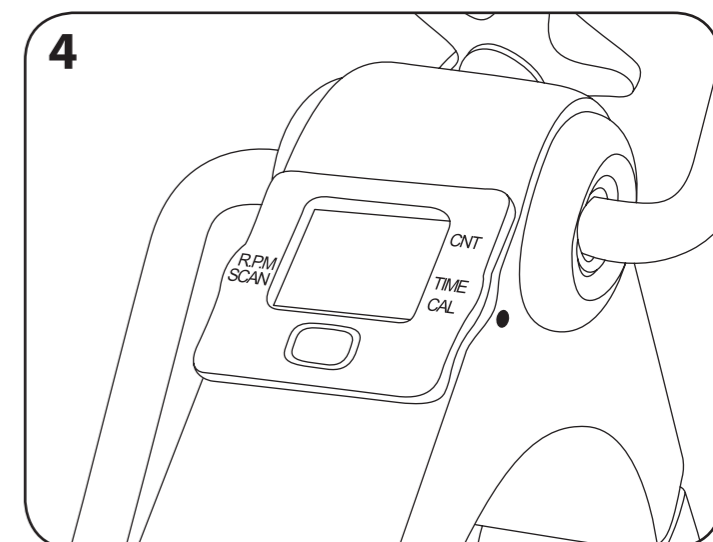Replace the screen unit in its socket.

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