

Thermoclay™

Natural Clay Compress / Comresse d'argile naturelle

INSTRUCTIONS

Hot Therapy

Place in microwave and heat in accordance with the instructions below. Do not heat until the compress has returned to room temperature.

Heating time

600 - 800 Watts 90 seconds	850 - 1000 Watts 60 seconds
-------------------------------	--------------------------------

Cold Therapy

Place in freezer for at least three hours.

Care

Clean surface of compress with a damp sponge and allow to air dry before use. Do not immerse in water. Store in a cool dry place. Dispose of in normal household waste if worn or damaged. If accidentally overheated, allow to fully cool in a safe place and do not touch.

! WARNING

**THIS PRODUCT CAN CAUSE BURNS.
AVOID PROLONGED CONTACT WITH THE SKIN.**

DO NOT OVERHEAT.

**TAKE CARE WHEN REMOVING FROM THE
MICROWAVE OVEN.**

**INSPECT AT REGULAR INTERVALS.
DISCARD IF FOUND WORN OR DAMAGED.
CLEAN ONLY IN ACCORDANCE WITH THE
INSTRUCTIONS ABOVE.**

DO NOT INGEST CONTENTS.

USE ONLY UNDER STRICT ADULT SUPERVISION.



Muscle pain
Spasm
Stiffness, etc.

Douleurs musculaires
Spasmes
Raideurs, etc.



Migraines
Sprains
Bruising, etc.

Migraines
Entorses
Ecchymoses, etc.

INSTRUCTIONS

Thérapie par la chaleur

Chauffer dans un four à micro-ondes selon les instructions ci-dessous. Chauffer la compresse seulement lorsque celle-ci est rendue à la température de la pièce.

Temps de chauffage

600 - 800 Watts 90 secondes	850 - 1000 Watts 60 secondes
--------------------------------	---------------------------------

Thérapie par le froid

Placer au congélateur pour au moins trois heures.

Entretien

Nettoyer la surface de la compresse avec une éponge humide et laisser sécher à l'air libre avant utilisation. Ne pas plonger dans l'eau. Entreposer dans un endroit frais et sec. Jeter dans les ordures ménagères si usée ou endommagée. En cas de surchauffe accidentelle, laisser refroidir complètement dans un lieu sûr et ne pas toucher.

! AVERTISSEMENTS

**CE PRODUIT PEUT CAUSER DES BRÛLURES.
ÉVITER LE CONTACT PROLONGÉ AVEC LA PEAU.**

NE PAS SURCHAUFFER.

**PRENEZ GARDE À LA TEMPÉRATURE DE LA
COMPRESSE LORSQUE VOUS LA SORTEZ DU
MICRO-ONDES.**

**INSPECTER RÉGULIÈREMENT. JETTER SI USÉE
OU ENDOMMAGÉE. NETTOYER SELON LES
INSTRUCTIONS CI-DESSUS.**

NE PAS INGURGER LE CONTENU.

**UTILISER SEULEMENT SOUS SUPERVISION
D'UN ADULTE.**

Rev1 0817 P/N 914-470820

LOT: