Basic Precautions

1. Before assembling and/or using the commode, please read and be sure to clearly understand this instruction sheet. Failure to do so could result in personal injury and/or damage to the commode.
2. Examine the commode for any damages that may have occurred during shipping. Do not use the commode if there is apparent damage. Please contact your dealer for repairs and/or for information on how to return the product.
3. Make sure that the toilet seat is at room temperature before installing it if it has been exposed to cold. Failure to do so could result in the breakage of the commode and/or possible injury.
4. Users with limited physical capability should be supervised and/or assisted while using the commode.

Maintenance

1. We recommend that you regularly verify that all the hardware parts are properly tightened.
2. This commode can be easily cleaned with mild liquid household cleaners. Avoid the use of harsh solvents, abrasive powders or scouring pads.
3. If a mechanical defect appears, contact your maintenance personnel for repair. Never use a defective commode.

Lifetime Limited Warranty

Your AMG Home Care Commode is warranted to be free from defects in material and workmanship under normal use. The warranty does not extend to non-durable components which are subject to normal wear and replacement.

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REV 1 0706 P/N 810-264

Assembly Instructions

IMPORTANT: Read the information and precautions on the back of this document before assembling / using this commode.

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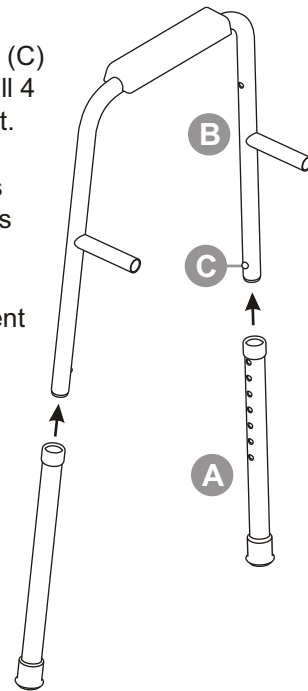
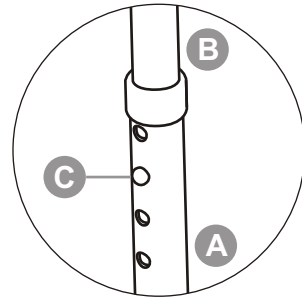
Assembly Instructions

Instructions d'assemblage



- 1** Push the side frames (B) into the extension legs (A) until the pop pins (C) snap firmly into place. Ensure that all 4 legs are adjusted to the same height.

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
- 2** Align and insert the side frame tubing (B) into the crossbar frame (D). Align the holes in the crossbar and side frame, then insert the E-clip (E) until it locks into place. Repeat for the remaining three alignments.

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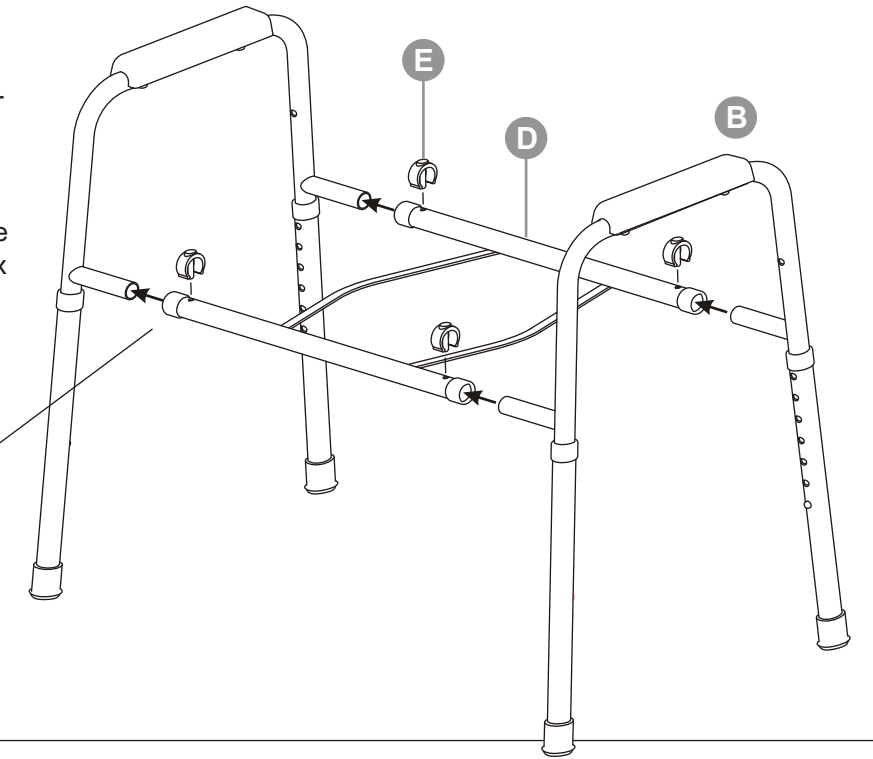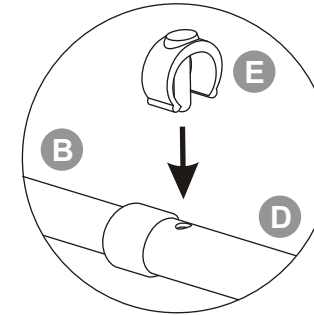
- 3** Align the holes in the backrest (F) with the side frame (B) holes. Insert the bolts (G) into the holes. Place the washers (H) and wing nuts (I) on the bolts as illustrated. Tighten the wing nuts (I) to secure.

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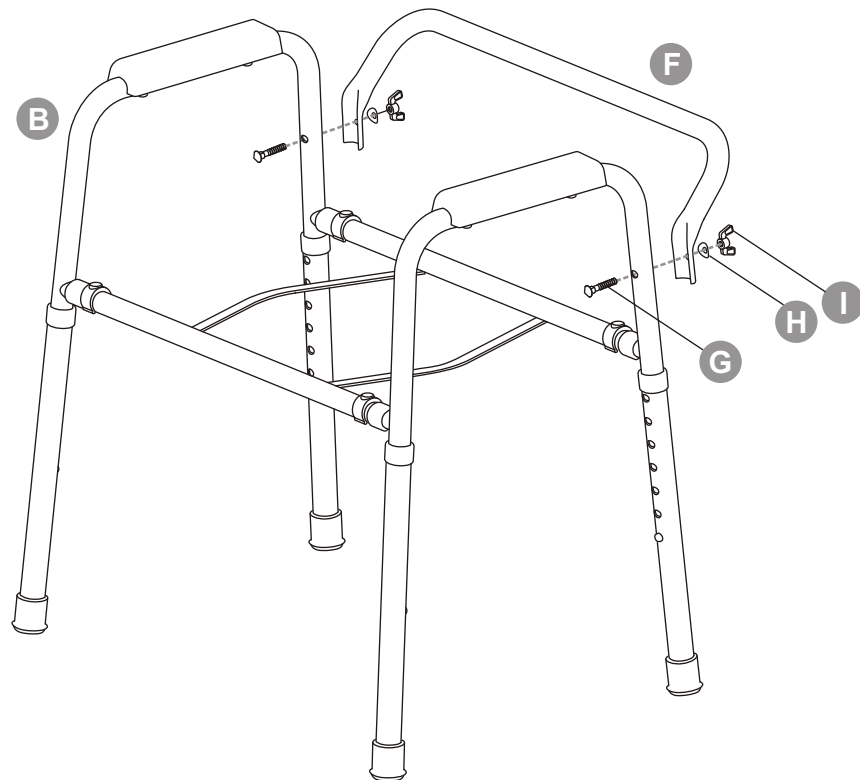
- 4** Install the commode pail (J) by sliding it over the pail holder of the crossbar.

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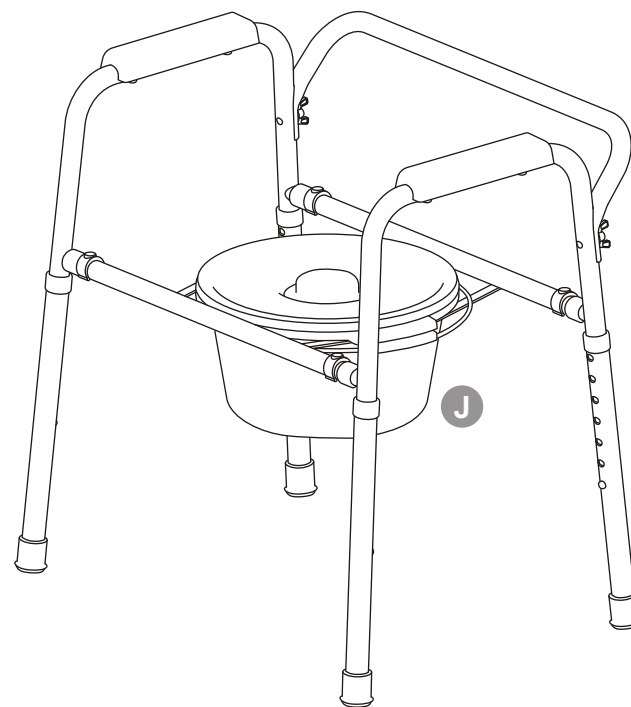- 5** Align the toilet seat clamps on the crossbar rails. Push down on the seat (K) until the clamps firmly snap into place. Repeat with the toilet seat cover (L), ensuring to align it correctly on either side of the seat before pressing to secure it into place.

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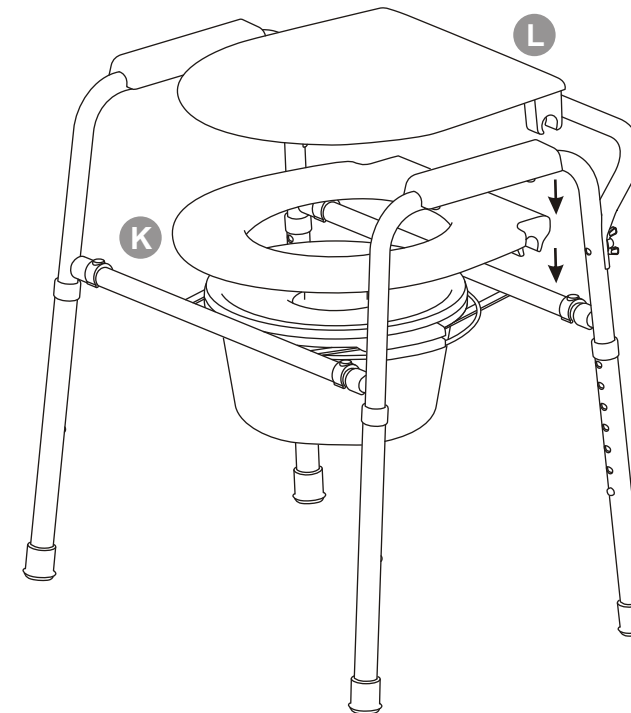- 6** To adjust the height of the commode after it has been assembled, turn the commode upside down with the pop pin holes facing you. Adjust each leg one at a time by pressing its pop-pin (A), sliding the leg to the desired hole location, and allowing the pop pin (B) to snap into place.

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AMG Medical Inc. Montreal, QC H4T 1V5