Lifetime Limited Warranty

Your A.M.G. Medical Inc. product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This product has been built to provide you with reliable service throughout its useful life. It meets strict quality standards designed to ensure the product can play its role, as expected. The Lifetime Limited Warranty is a sign of our commitment to deliver the value that we have promised to you.

This warranty does not cover device failure due to owner alteration, misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

For warranty service and general inquiries in Canada and the U.S., call our customer service team at 1-800-363-2381.

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To learn more about this or any other **A.M.G.** product, please contact your **A.M.G. Medical Inc.** representative or call toll-free: **1-800-363-2381**.

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A.M.G. Medical Inc.
Montréal, Québec
H4T 1V5 Canada
www.amgmedical.com

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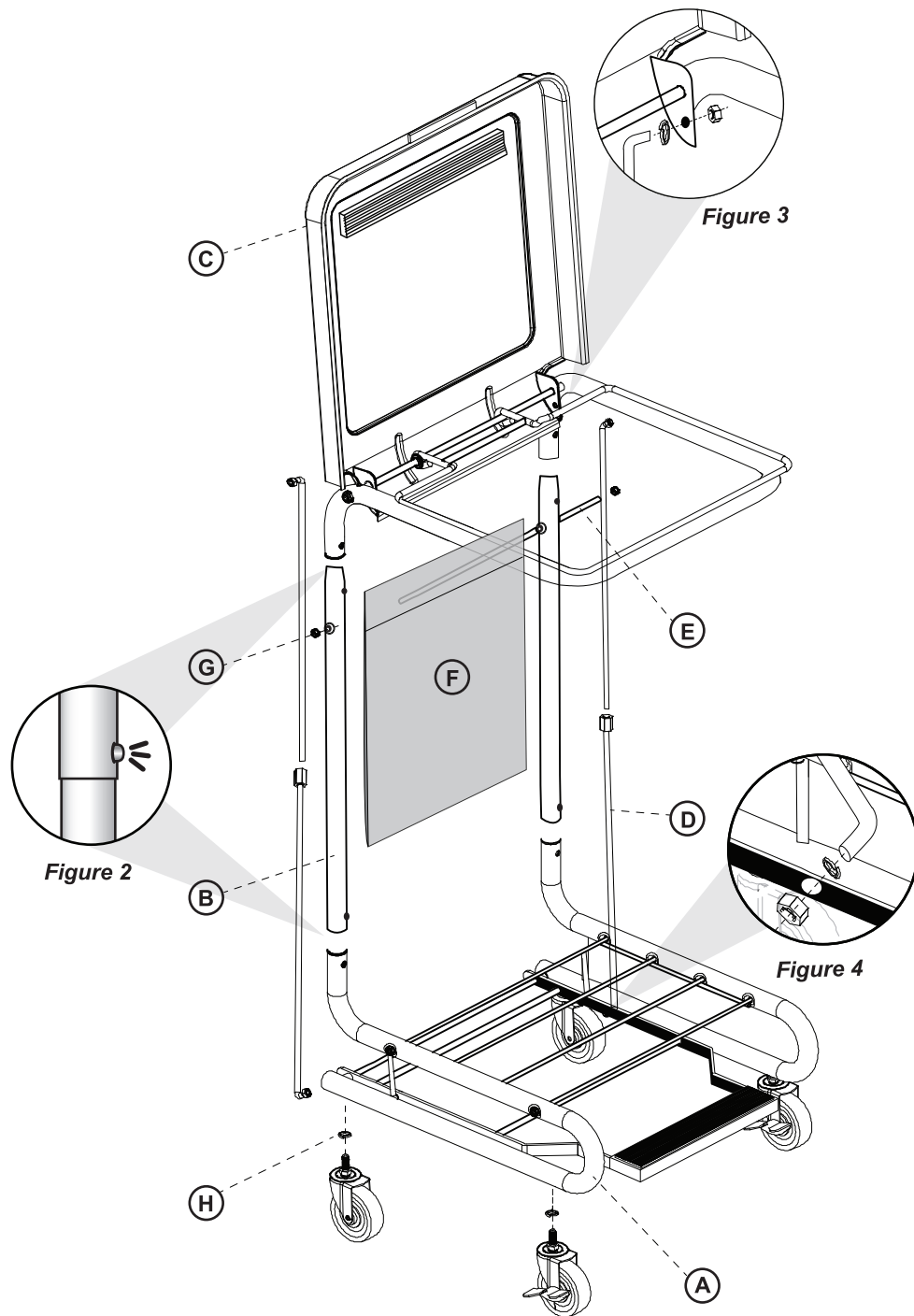
Figure 1

Step 1

Fasten locking casters to the front end of base. Fasten non-locking casters to the rear end of base. Use (1) each of the 3/8" lockwasher (H).

Step 2

Insert the (2) tubular uprights (B) until their snap pins interlock with the pin holes in base (A). (Figure 2)

Step 3

Attach the tubular top frame and lid subassembly (C) to the tubular uprights (B) until the snap pins interlock with the hole in the tube. (Figure 2)

Step 4

Assemble bag holder rod (E) through one of the tubular upright (B). Insert the vinyl pouch (F) and through the other tubular upright. Secure ends using the (2) 1/4" lock nuts (G).

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Step 6

Assemble one end of each connecting rod (D) through the holes in lid. (Figure 3) The rod ends must face outward. Secure the ends with the 1/4" lock nuts and plastic washer.

Step 7

Assemble the opposite ends of the rods into the foot pedal. (Figure 4) The rod ends must face inward. Secure the ends with the 1/4" lock nuts and plastic washer.

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